

Porcelain Primer

17-411-100

RECOMMENDED INDICATIONS:

Optimal adhesion between metal – composite or ceramic - composite is provided by a chemical bond. Used to Silanization of silicized ceramic or metal restorations prior to cementation.

CONTRAINDICATIONS:

May cause drowsiness or dizziness

COMPOSITION:

3- Glycidoxypropyltrimethoxysilane

Ethyl alcohol

PRODUCT PRESENTATION

15ml Bottle # 17-411-100

INSTRUCTIONS FOR USE:

- 1- Isolate: A dry operating field is essential for the Silanization procedure. Liquid dam (17-313-201) is the preferred method of isolation.
- 2- For each use, freshly dose porcelain primer in a clean, grease-free dish. Immediately after use reseal the bottle containing the volatile solution. Use a disposable brush to apply the silane solution to the surface(s).
- 3- Allow the volatile silane solution to dry for 30 sec.
- 4- The silanized surfaces must be completely dry before applying the composite.
- 5- After drying, proceed the composite Comply with the Instructions for Use of the individual products.

WARNING:

Porcelain primer is highly flammable. Avoid static discharge. Keep away from heat, sparks, open flame, pilot lights, and other sources of ignition.

USE AND STORAGE



Store away from sunlight



Observe storage temperatures



Protect from moisture



SHELF LIFE – 2.5 Year

EMERGENCY MEASURES:

Eye contact – flush with copious amounts of water. Seek medical attention if irritation persists.

Ingestion – Rinse mouth. Do NOT induce vomiting. Absorb spillage to prevent material damage

Skin contact – Wash with soap and water

Inhalation – Remove person to fresh air

Limitation of Liability:

Except where prohibited by law, IOS will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Caution: Federal law restricts this device to sale by, or on the order of, a dentist (or dental professional)

International Orthodontic Services
12811 Capricorn St. Stafford, TX 77477 USA.
T: +1 (832) 342 9487 F: +1 (281) 340 4794
www.iosortho.com info@iosortho.com